


1232 Kingston Road, Scarborough, Ontario M1N 1P3
Tel: 416-691-4167 Toll-Free: 1-877-842-4827 Fax: 416-691-8112
E-mail: info@accu-metrics.com Web: www.accu-metrics.com

DNA-VIAN BIRD SEX DETERMINATION FEATHER COLLECTION INSTRUCTIONS

Feathers for avian DNA testing should be freshly plucked, not naturally molted. DNA is contained at the base of the feather, which originates just below the surface of the skin. It is important to pluck each feather individually and close to the skin. Use a pair of sanitized tweezers, if necessary—particularly with smaller birds. Avoid plucking feathers from the opposite end or plucking in clumps, as this may cause the shafts to break off above the skin, rendering them unusable. *See illustration*.

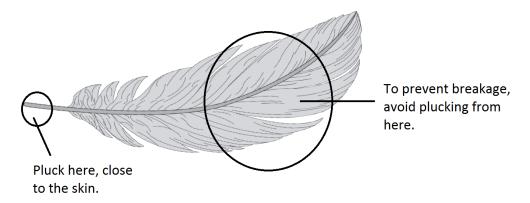

Before taking any feather samples, make sure that your hands are clean. If you are collecting samples from multiple birds, wash hands between each collection. Avoid touching the ends of the feather shafts. Follow the steps below to ensure proper feather collection for the avian DNA test.

1. Label Each Envelope or Bag

Before you start collecting the feather samples, label each envelope or resealable plastic bag with the submitter's name, species of bird collected, and a Bird ID. The Bird ID can be the bird's name, band number, or cage number. Use the same Bird ID on the Avian DNA Sexing Submission Form so that you can match the final DNA results to the correct bird.

2. Pluck Chest Feathers

With the bird restrained, pluck 4-5 chest feathers. Blood or primary feathers are not required. Additional feathers may be required for smaller birds, such as finches.

3. Place Feathers in the Labeled Envelope or Bag

Place the feathers in the envelope or plastic bag that was labeled in Step 1. Avoid touching the ends of the feather shafts. Each envelope or bag should contain feather samples from only **ONE** bird.